

### ONE YEAR LIMITED WARRANTY

**WARRANTY:** The company warrants this product to the original purchaser or gift recipient, to be free from defects in workmanship and materials under normal use and services, for a period of one year from the date of purchase.

**LIMITATIONS:** ALL WARRANTIES IMPLIED BY STATE OR PROVINCIAL LAW, INCLUDING THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE, ARE EXPRESSLY LIMITED TO THE DURATION OF THE LIMITED WARRANTIES SET FORTH ABOVE. Some jurisdictions do not allow limitation on how long an implied warranty lasts, so the above limitation may not apply to you. IN NO EVENT SHALL THE COMPANY BE LIABLE FOR ANY INCIDENTAL OR CONSEQUENTIAL DAMAGES, LOSS OF PROFIT OR MEDICAL EXPENSES CAUSED BY ANY DEFECT, FAILURE, MISUSE, OR MALFUNCTION OF THE PRODUCT. Some jurisdictions do not allow the exclusion of limitation by incidental or consequential damages, so the above limitation or exclusion may not apply to you. The company will not be responsible for damages or losses, direct or indirect, caused by misuse, abuse, accident, negligence, conditions of transport or storage, or failure to follow instruction. The company will not be responsible for any statements that are made or published, written or oral, that are inconsistent with this written warranty, or which are misleading or inconsistent with the facts as published in the literature or specifications furnished by the Company. This warranty gives you specific legal rights, and you may also have other rights which vary from state to state.

Warranty Claims Procedure: Any product that proves defective within the warranty period will, transportation charges pre-paid, be repaired or replaced at the option of the Company, free of charge.



## Equine TMJ Hood Therapy System

Read the contents of this booklet  
carefully before product use



Phone 1-800-975-0253 if you have questions not answered in these instructions.

## **INSTRUCTIONS FOR USE - Please read carefully**

### **General Step by Step Guidelines:**


1. Secure the horse in cross ties.
2. Before treatment, brush dirt away from the head and neck of the horse.
3. Pre-warm the TMJ Hood system on the high setting for a few minutes before placing on the horse's head.
4. Fasten the TMJ Hood comfortably on the horse using the Velcro Straps.
5. Use the high setting for first 10 minutes of therapy.
6. After the initial ten minutes, turn the switch to the low setting and maintain this setting until the therapy session is complete.
7. Disinfect the inside of the TMJ Hood with a mild disinfectant after use and allow it to dry before storing in a clean place.
8. If necessary sponge off the horse's head after treatment.

### **Important Safety Instructions**

- Never use the TMJ Hood along with any type of liniment, salve or ointment.
- Do not put the TMJ Hood on the horse and leave unattended. Check hide under TMJ Hood frequently.
- Do not immerse the TMJ Hood cover in water unless the heat inserts and wiring harness have been removed.
- Do not use pins or other metallic objects to hold the TMJ Hood cover in place.
- Never use the heat inserts without yellow cover in place.
- Periodically examine the heat inserts (PVC). Return the TMJ Hood for repair if the heat inserts show any sign of deterioration.
- Do not fold or bend the heat inserts.
- Do not pull the TMJ Hood by the cord or use the cord as a handle.

## **TMJ Hood Assembly**

**Both cords from the TMJ heat inserts are run through this channel and then branch out left and right here**

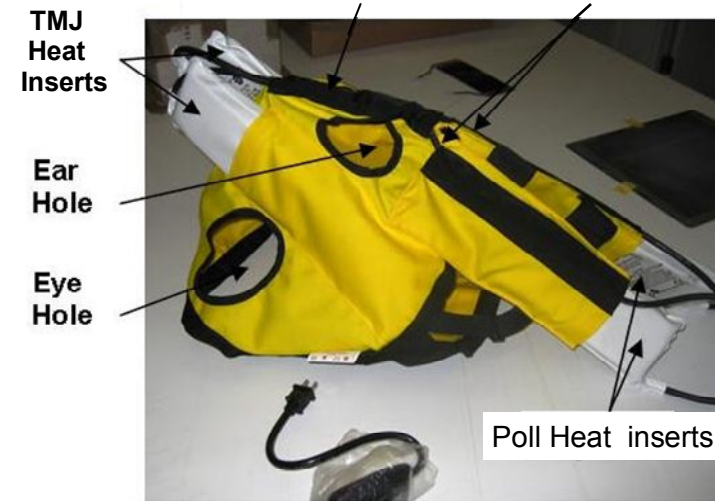

- There are 4 heat inserts in the TMJ Hood System.
- Two heat inserts are positioned along side the TMJ (temporomandibular joint) and two heat inserts along the poll.
- The “hard side” of the heat inserts should be placed inside the TMJ Hood cover so that the hard side is facing towards the horse when the TMJ Hood is in place.
- Each heat insert plugs into the harness which runs along a top spine that branches into two veins to accommodate the heat inserts used for treating the poll area.