

### ONE YEAR LIMITED WARRANTY

**WARRANTY:** The company warrants this product to the original purchaser or gift recipient, to be free from defects in workmanship and materials under normal use and services, for a period of one year from the date of purchase.

**LIMITATIONS:** ALL WARRANTIES IMPLIED BY STATE OR PROVINCIAL LAW, INCLUDING THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE, ARE EXPRESSLY LIMITED TO THE DURATION OF THE LIMITED WARRANTIES SET FORTH ABOVE. Some jurisdictions do not allow limitation on how long an implied warranty lasts, so the above limitation may not apply to you. IN NO EVENT SHALL THE COMPANY BE LIABLE FOR ANY INCIDENTAL OR CONSEQUENTIAL DAMAGES, LOSS OF PROFIT OR MEDICAL EXPENSES CAUSED BY ANY DEFECT, FAILURE, MISUSE, OR MALFUNCTION OF THE PRODUCT. Some jurisdictions do not allow the exclusion of limitation by incidental or consequential damages, so the above limitation or exclusion may not apply to you. The company will not be responsible for damages or losses, direct or indirect, caused by misuse, abuse, accident, negligence, conditions of transport or storage, or failure to follow instruction. The company will not be responsible for any statements that are made or published, written or oral, that are inconsistent with this written warranty, or which are misleading or inconsistent with the facts as published in the literature or specifications furnished by the Company. This warranty gives you specific legal rights, and you may also have other rights which vary from state to state.

Warranty Claims Procedure: Any product that proves defective within the warranty period will, transportation charges pre-paid, be repaired or replaced at the option of the Company, free of charge.



## Equine Hood Therapy System

Read the contents of this booklet  
carefully before product use



Phone 1-800-975-0253 if you have questions not answered in these instructions.

## INSTRUCTIONS FOR USE - Please read carefully

### General Step by Step Guidelines:

1. Secure the horse in cross ties.
2. Before treatment, brush dirt away from the head and throat of the horse.
3. Pre-warm the Hood system on the high setting for a few minutes before placing on the horse's head.
4. Fasten the Hood comfortably on the horse using the Velcro Straps.
5. Use the high setting for first 10 minutes of therapy.
6. After the initial ten minutes, turn the switch to the low setting and maintain this setting until the therapy session is complete.
7. Disinfect the inside of the Hood with a mild disinfectant after use and allow it to dry before storing in a clean place.
8. If necessary sponge off the horse's head after treatment.

### Important Safety Instructions

- Never use the Thermotex Hood along with any type of liniment, salve or ointment.
- Do not put on the horse and leave unattended. Check hide under the Hood frequently.
- Do not immerse the Hood cover in water unless the heat inserts and wiring harness have been removed.
- Do not use pins or other metallic objects to hold the Hood cover in place.
- Never use heat inserts without yellow cover in place.
- Periodically examine the heat inserts (PVC). Return the Hood for repair if the heat inserts show any sign of deterioration.
- Do not fold or bend the heat inserts.
- Do not pull the Hood by the cord or use the cord as a handle.

## Hood Assembly



Top Heat Insert

Heat Insert Beneath Jaw

- There are two heat inserts in the Hood System
- One heat insert is positioned on the top of the forehead, midway between the eyes and the other heat insert is positioned mid centre beneath the jaw.
- The “hard side” of the heat insert should be placed inside the Hood cover so that the hard side is facing toward the horse when the Hood is in place
- Each heat insert plugs into the harness which runs up through the top of the system

Top Heat Insert



Heat insert beneath jaw

Top of Hood with heat insert in place and harness running down the neck

Harness Channel



Heat Insert Placement in lower jaw pocket